

Bermedi ProFix X-Ray Fixer for Roller Machines



Description

- X - ray fixer for all customary developing machines with roller transport and for spray-developing machines with a fixage time by 15 s or more
- Because of its excellent hardening properties and the fast working fixer components the product is suitable for use in fast going machines
- application in conjunction with Developers like Bermedi ProDev
- liquid concentrate
- two – part

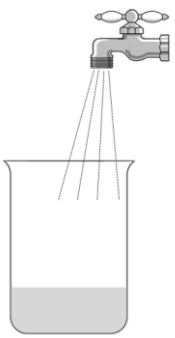
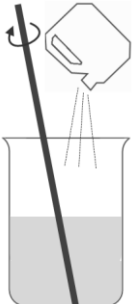
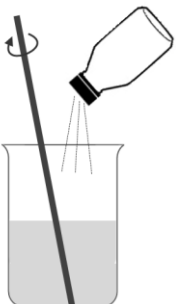
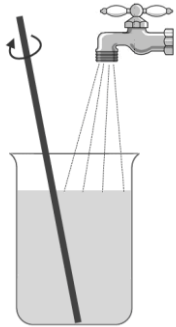
Replenishment

- For processing of 1 m² X-ray film 600 to 750 ml fixer are necessary at standard conditions
- Generally the replenishment occurs automatically by addition of the solution in dependence of the processing film

Mix Instructions

preparation X - ray fixer replenisher:

- First dissolve part 1 completely in water, afterwards add part 2 under permanent stirring
- When using an adapter Bermedi ProFix is also suitable for Agfa- and Kodak – Mixer

Water (25-30 °C)	Part 1	Part 2	Fill up with water to
			
14,0 l	5,0 l	1,0 l	20,0 l

Storage and Handling

- Until the preparation the concentrate has to be kept in the closed original package and stored at temperatures of 10 to 25 °C
- Bermedi ProFix contains components which can cause irritations in case of contact with the skin.
- All users of photographic processing solutions should exercise the greatest care to avoid the chemicals contacting the skin or eyes. Always wear solution resistant gloves.
- In case of contact with the eyes it has to be rinsed thoroughly with plenty of running cold water by opened eyelid and a medical doctor has to be consulted
- The disposal of used photographic processing solutions has to be carried out in accordance with the legal regulations

Assortment

Product-ID	Product Name	Mix Size for	Concentrate	Capacity
031015	Bermedi ProFix	2 x 20 l	Part 1: 2 x 5,0 l Part 2: 2 x 1,0 l	50 – 65 m ²